Medizid[®] Alpha Flowpack

Our ready-to-use, alcohol-free hygienic wipes for disinfecting and cleaning surfaces and objects

Product type

Medizid® Alpha disinfectant wipes (Flowpack) are ready-to-use, aldehyde and alcohol-free hygiene wipes for cleaning and disinfecting small surfaces and objects. Ideal for use between treatments and interventions in healthcare facilities. Bactericidal, yeasticidal, virucidal effect against all enveloped viruses including HBV / HIV. Medizid® Alpha disinfectant wipes (Flowpack) are impregnated with the very fast acting liquid of non-alcohol based Medizid® Alpha.



For all surfaces and objects, in particular for device surfaces and objects near the patient in clinics, surgeries, ambulances and care areas.

Application notes

Ready to use. Completely wet objects, surfaces, allow to dry, if necessary wipe. Only use damp cloths for disinfection.

Shelf life

The minimum shelf life of Medizid® Alpha disinfectant wipes (Flowpack) is 3 years (see label). To prevent the premature drying of wipes, close the packaging immediately after use. Shelf life / expiry after opening: 28 days. The products must only be stored in the original sealed container, contamination must be avoided while removing product, and the products must be kept and stored in accordance with standard storage conditions (protected from damp, direct sunlight, heat and frost) at a temperature of between 5 and 25°C.

Composition

In 100 g of impregnation solution are included: 0.2 g Polyhexamethylene biguanide hydrochloride, 0.64 g Didecyldimethylammoniumchloride.





Product properties

- ready to use
- Fast and comprehensive, effective already after 60 seconds
- comprehensive bactericidal and yeasticidal effect
- effective against all enveloped viruses:
 BVDV / Vaccinia (incl. HCV / HBV / HIV)
- effective against non-enveloped viruses: Rota, Polyoma / SV40
- pleasant smell
- very good material compatibility even with sensitive materials, such as acrylic glass
- 100% cellulose